FLU-STONE

Descripción

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FLU-STONE

Description

Model base plaster. Type IV, high quality ISO 6873

Uses

Very fluid gypsum; it can be poured into the base former without vibration.

Recommended for all pin systems of sectional die stone.

Advantages

Once set, the obtained surface is very compact and smooth, allowing a good separation between the dies.

Free of bubbles even without vibration and with manual mixing.

Due to its low expansion, the obtained models don't have tension caused by differences of expansions.

Intense colours that contrast with the model.

Physical characteristics

Working time: 6 min
Initial setting: 8 min (Vicac test according to ISO 6873)
Final setting: 11 min (Vicac test according to ISO 6873)
Setting time: 30 min
Setting expansion (2h): 0.05 %
Compressive strength (1h): > 40 MPa

Working instructions

Powder/water ratio: 100 g / 24 ml
Working temperature: 20-23°C
Mixing: Manual, 60 sec
Vacuum 30 sec

Add the powder to the demineralised water following the recommended ratio.

When this mixing time has finished, the mix must have fluid consistency, flowing easily on the base model even without vibration.

Cautions

1.- USE CAUTIONS:

Use always-clean instruments

Do not mix the product in a container where investments have been mixed.

Use always demineralised water to do the mix. The water from the public mains can affect the correct setting process of the gypsum, making it faster or slower

2.- SAFETY CAUTIONS:

Do not inhale the powder. It's recommended the use of a protective mask.

Keep away from children.

Handle the product in a well-ventilated place.

The handling of this product has to be done only by qualified people

In the event of ingestion, consult a doctor.

For further information, see the safety data sheet for the product.

Storage and expiry

The stability of the product is 24 months respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

Storage: Keep the packages tightly closed in a dry place away from moisture.

After each use the package must be closed to prevent the moisture absorption.

Supply

Terra	Blue	Intense Blue	Indian Red	
1130-067	1130-061	1130-068	1130-069	6 kg (3 x 2 kg)
1130-M17	1130-M11	1130-M18	1130-M19	Test pack 100g

Additional information

The instructions that appear on this sheet are based on our current knowledge and are given only as information without that implies a guarantee from Protechno.

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